

## **DRE Versailles Series** Mobile Surgery Tables

# Complete multi-purpose mobile operating system



Designed for



*Pre-op*



*Transport*



*Surgery*



*Recovery*

***The mobile surgery table transports patients from pre-op through recovery, eliminating patient transfers and increasing efficiency and safety***

# The DRE Versailles P100 mobile surgery table goes beyond the O.R.

TRANSPORT



SURGERY



The DRE Versailles P100 is a complete surgery solution that can **move patients from the pre-operative area to the operating room and then to the recovery area** — with no need to transfer the patient from a stretcher to a surgery table and back.

The DRE Versailles P100 does it all, increasing throughput by incorporating **stretcher mobility** into a **multi-purpose surgery table!**

RECOVERY



PRE-OP





Powered patient trolley/operating table is a total surgery and transport solution!

**Features**

- Four remote-activated powered functions.
- Excellent stability for surgery procedures.
- 550-lb. (250-kg) weight limit.
- Traversing patient platform.
- Superior surgical access.
- 100% C-arm coverage.
- Lateral tilt ( $\pm 12^\circ$ ).

Headrests available for many surgical procedures.

Low trolley height helps patients easily get on and off the tabletop.

**STRETCHER MANEUVERING**



Cotsides fitted for patient comfort and safety.

Unique retractable steering mechanism engaged for maximum maneuverability.



Removable head section



Push button release and self-locking cotsides



Push button rotating cotsides



Push button release and self-locking leg section




K8 pressure care mattress



Removable lightweight leg section

**SURGERY TABLE POSITIONING**

Positions include Trendelenburg, Reverse Trendelenburg, lateral tilt, head rest articulation, and more.

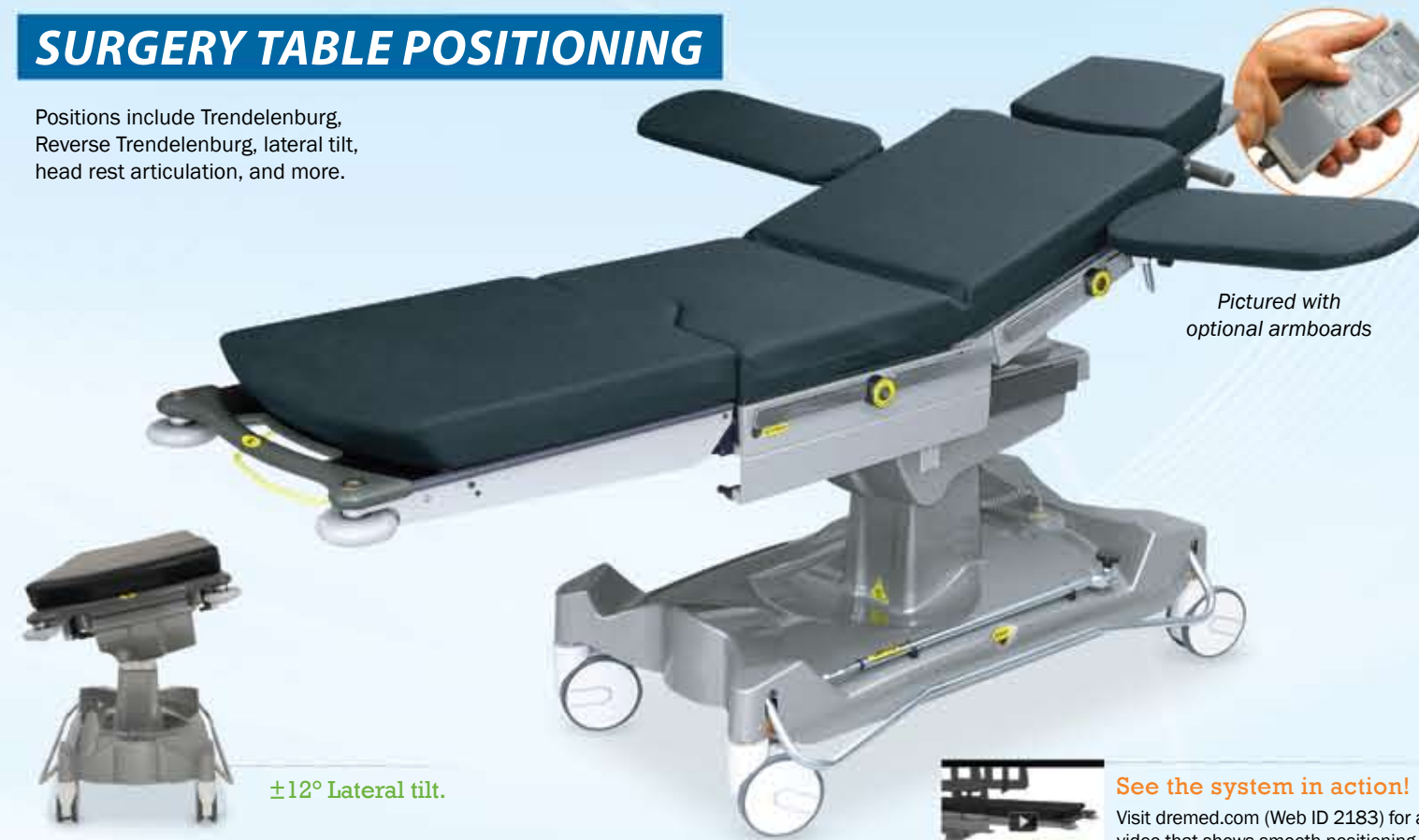

Pictured with optional armboards

$\pm 12^\circ$  Lateral tilt.



See the system in action!

Visit [dremed.com](http://dremed.com) (Web ID 2183) for a video that shows smooth positioning and accessories for the mobile surgery table!

**Optional Accessories**



Lateral Support Complete (Requires Clamp)



Carter Braine Arm Support Pad (Requires Clamp)




Pelvic Support Complete (Requires Clamp)



Lithotomy Poles Complete (Requires Clamp)



Square Arm Table Complete (Requires Clamp)

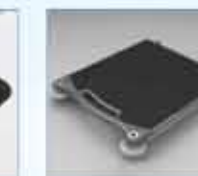

Articulating Removable Leg Section



Shoulder Arthroscopy Support



**Shoulder Arthroscopy**

Patient platform traversed to foot end for access.

Optional arthroscopy backrest fitted.

Backrest articulation for comfortable positioning.



**Ophthalmology**

Patient platform traversed to head end for access.

Ophthalmic head section fitted.

Backrest articulation for comfortable positioning.



**ENT & Oral Surgery**

Patient positioned on standard articulating head section.

Tabletop can be traversed to head end for superior access.

Backrest articulation for comfortable positioning.




**Gynecology**

Lightweight leg and head sections removed for access.

Patient platform traversed to foot end for access.

Fitted with optional powered leg supports.

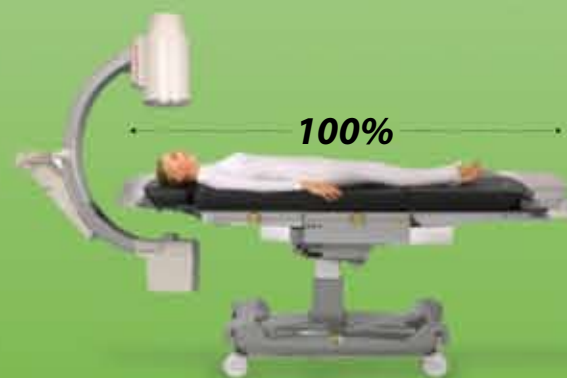

Unique traversing top allows for 100% C-arm access



**Accessory Trolley**

Complete storage solution for Versailles accessories and general O.R. accessories

Also available **DRE Versailles M75 Mobile Surgery Table**

The manual configuration provides excellent maneuvering and easy-to-use hand-operated functions.

**Features**

- » Mechanical functions
- » Lightweight and easy to maneuver
- » 350-pound (160 kg) weight limit
- » Increased surgical access
- » 75% C-arm access



**ENT/Dental**

Patient positioned on standard articulating head section; backrest articulation for comfortable positioning.



**Gynecology**

Lightweight leg section removed for access and head section removed for anesthesiologist access; fitted with optional powered leg supports.



# DRE Versailles Series Mobile Surgery Tables

Specifications	Powered (V100)	Manual (M75)
Powered Functions	Yes	No
Lateral Tilt	Yes	No
Head End Traverse	Yes	No
Foot End Traverse	Yes	No
K8 Pressure Care Mattress	Yes	Yes
Removable Ultra-Lightweight Leg Section	Yes	Yes
Removable and Rotating Cotsides	Yes	Yes
Push Button Release and Self-Locking Controls	Yes	Yes
5th Wheel Steering Mechanism	Yes	Yes
Advanced Four-Caster Braking System	Yes	Yes
'E' Size Oxygen Cylinder Trough (Side Loading)	Yes	Yes
Transfusion Pole with Mounting Position	Yes	Yes
150 mm Casters	Yes	Yes
Height (min)	23.8" (605 mm)	22.6" (575 mm)
Height (max)	39.5" (1,005 mm)	36.8" (935 mm)
Trolley length	82.7" (2,100 mm)	82.7" (2,100 mm)
Side Rail Width	25.8" (655 mm)	25.8" (655 mm)
Cotside Width	30.3" (770 mm)	30.3" (770 mm)
Brake Width (Brakes off)	33" (835 mm)	33" (835 mm)
Mattress Length	80.3" (2,040 mm)	80.3" (2,040 mm)
Mattress Width	23.6" (600 mm)	23.6" (600 mm)
Mattress Depth	3" (75 mm)	3" (75 mm)
Base Length	47" (1,190 mm)	41.5" (1,055 mm)
Trendelenburg	20°	20°
Reverse Trendelenburg	12°	N/A
Lateral Tilt	+12°/-12°	N/A
Backrest Articulation	0 - 65°	0 - 80°
Head Section Articulation	+30°/-30°	+30°/-30°
Head End Traverse	9" (230 mm)	N/A
Foot End Traverse	9" (230 mm)	N/A
C-arm Access	100%	75%
Max. Patient Weight Limits	550 lbs (250 kg)	352 lbs (160 kg)
Trolley Weight	352 lbs (160 kg)	275 lbs (125 kg)
Lightweight Leg Section Weight	13.2 lbs (6 kg)	13.2 lbs (6 kg)

Specifications subject to change without notice.

## Optional Features

- » Spare Transfusion Pole
- » Sculpted Mattress for Head Support
- » Ophthalmic Head Support
- » Removable Shoulder Arthroscopy Backrest
- » Padded Cotside Covers - Gray
- » Padded Cotside Covers - Child Print
- » Articulating Removable Leg Section
- » Removable Leg Section with Full Length Side Rail
- » Pressure Mattress Complete
- » Accessory Trolley
- » Accessory Trolley Spare Trough
- » Spare Powered Trolley Hand Control
- » Spare Powered Trolley Charging Cable
- » CD Side Cylinder Support Bracket



Patient Transport



Ophthalmology



Arthroscopy



Gynecology



C-arm Coverage



TOLL-FREE 1-800-477-2006 ONLINE [www.dremed.com](http://www.dremed.com)  
1800 Williamson Court Louisville, KY 40223 USA